

Plate Storage Conditions

Opened plates are sensitive to storage conditions. Microplates must be stored at $5^{\circ}\text{C} \pm 3^{\circ}\text{C}$. In addition, it is essential to immediately close the aluminium envelope after opening. The envelope must be sealed hermetically to guarantee the conservation of the remaining strips and stored at $5^{\circ}\text{C} \pm 3^{\circ}\text{C}$. The simple fold-over of the edges is insufficient to ensure the sealing. The end of the bag should be cut with scissors and closed hermetically along the entire length (see photos below).

